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
## Digital alcohol breath tester „SILVER”


### INSTRUCTIONS FOR USE

1. LCD display
2. Power button
3. Breath inhaler
4. Battery compartment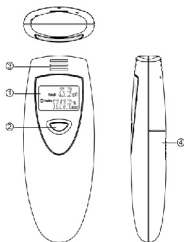


### Features

1. Digital alcohol test with audible alert.
2. Range:
  - 0,00-0,19 % BAC
  - 0,01-1,9 g/l (0,0 - 1,9 ‰)
3. Quick response and resume.
4. Auto power off.
5. Use 2x AAA alkaline batteries.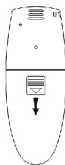

### Installing battery


1. Slide open the battery compartment lid from the back of the unit.
2. Insert two AAA alkaline batteries with correct polarity as indicated.
3. Replace the battery compartment lid.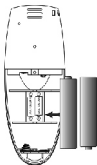

## How to use the alcohol tester

### Warm up

1. Press and hold the <POWER> button for 1 second, the alcohol tester LCD will turn on with one beep sound.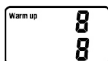


2. The “warm up” symbol is displayed, and the 10 to 00 countdown is started that indicate the tester is in warm-up mode.

### Note:

*It always has a large number of gas & impurity adsorbed on the surface of alcohol tester, when first time taking out the tester from the packaging, or after long time no using for the tester, it need more time for cleaning and warm up the sensor, you should repress the power button and retry several times if possible.*

## Exhale Test

1. The "Blow" symbol is displayed when warm-up is succeed, and the 10 to 00 countdown is started that indicate the tester is in testing mode.

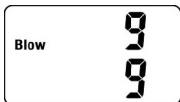2. To perform the Test, the person should take a deep inhale(breath), then, exhale with some strength by mouth into the breath inhaled for about 4-5 seconds without interruption.
3. Read the test result from the alcohol tester LCD.
4. The "Caution" symbol is displayed if the alcohol concentration is the level:  
0.02% BAC (or 0.2g/l) to 0.05% BAC (or 0.5g/l)


5. The "Danger" symbol is displayed with beep audible alert if the alcohol concentration is the level:  
Over & equal 0.05% BAC (or 0.5g/l)
6. If no blow into the breath inhaled at the test mode, the LCD will display "0.00 %BAC & 0.0g/l" value.

*Note: To obtain more accurate result, manufacturer recommend you to check repeatedly 2 or 3 times.*

### Auto power off

After the result be display for about 10 seconds, the "OFF" symbol will appear for 2-3 seconds and the alcohol tester be turned off.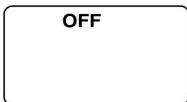


### Replace battery

The LCD flickering indicates low power of batteries, please replace two "AAA" alkaline batteries.

### Fixing the hand strap

1. Insert the smaller loop into the hole in the housing.
2. Thread the longer loop through the eyelet and pull through.

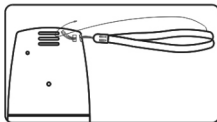## Specifications

Sensor	semiconductor oxide alcohol sensor
Range	0.00%-0.19% BAC (0.0-1.9g/l)
Power input:	2×1.5V AAA battery (not included)
Operating time	warm up: 10s, response: <5s
Model	01 900

## Precaution

- When after drinking, it is recommended to last after 20 minutes, it is because 20 minutes is approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.
- For accurate result, do not repeat next testing within 3 minutes after last time.
- Do not smoke and eat for 20 minutes prior testing.
- Do not use chemicals to clean the tester directly.
- Do not use any excitant substances near the tester, such as paint, insecticides, alcohol.
- Do not place the tester in the closed environment with contaminated air.
- Modifying or tampering with the unit's internal components may cause a malfunction.
- Each individual's body responds differently to alcohol. A person may become legally impaired to drive at a point below the legal blood alcohol limit.

**The result of this alcohol gives BAC for reference only, manufacturer does not to take any legal responsibility.**

**Instructions and information on management of used packaging**

Dispose of used packaging material to a designated municipal waste facility.

**Disposal of used electrical and electronic equipment**

This symbol on products or accompanying documents means that used electrical and electronic products should not be treated as household waste. For proper disposal and recycling, take these products to designated collection points. Alternatively, in some European Union states or other European countries you may return your products to

your local retailer when buying an equivalent new product. Correct disposal of this product will help save valuable natural resources and prevent any potential negative impacts on the environment and human health, which could otherwise be caused by improper waste disposal. For more details, please contact your local authorities or the nearest collection site. The improper disposal of this waste can be in accordance with national regulations for fines.

**For business entities in the European Union**

To dispose of an electrical or electronic device, request the necessary information from your seller or supplier.

**Disposal in other countries outside the European Union**

This symbol is valid in the European Union. To dispose of this product, request the necessary information about the correct disposal method from local government departments or from your seller.

This product meets all the basic requirements of EU directives that apply to it.

Changes to the text, design and technical specifications are subject to change without prior notice and we reserve the right to change them.

**Compass CZ s.r.o.** Pražská 17,  
250 81 - Nehvizdy, CZ, e-mail: [info@compass.cz](mailto:info@compass.cz)

*revize textu 2/2017*